



Instruction for Use - MPR Hand Traction Assembly

MPR-9037

Instructions for Use

Place the patient's fingers in the finger traps (either nylon or stainless steel traps may be used).

If using stainless steel finger traps, fingers may be wrapped in sterile dressing to reduce skin trauma.

Once fingers are secure in the traps, place the loop of the finger trap in the appropriate clip on the Hand Traction System. The Hand Traction System may be suspended from an IV pole or traction system.

Check that the Hand Traction System is tightly secured to the IV pole or traction system before patient use.

If the Hand Traction System, chain or attaching clips are damaged, **DO NOT USE**. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

Cleaning and Sterilization

Device must be cleaned prior to sterilization. Using any non-acidic, general purpose cleaner suited for standard surgical instruments, following the cleaner manufacturer instructions. Hand Traction Assembly should then be autoclaved.

Cycle Type: Pre-Vac Steam Sterilization
Full Cycle Instructions

Single wrap in standard central supply wrap
Sterilization Temperature: 132° C
Sterilization Exposure Time: 4.0 Minutes