



Cleaning and Sterilization for MPR Leg Holders | 5371 & 5372

Covers MPR legholders
5371 & 5372

Recommended hand cleaning instructions

Soak the product in an enzyme solution. Follow the manufacturer's directions for dilution and soaking time.

Impacted material should be scrubbed out; however, steel wool or abrasive soap cleaners should not be used as they may scratch the surface finish of the device.

Rinse the product in warm tap water.

Soak or wipe the product down with a hospital approved and/or EPA approved germicide.

Rinse the product in warm tap water.

Wash the product with an instrument detergent up to a pH of 9.0.

Rinse the product in warm tap water.

Dry thoroughly and wrap.

Note: : If necessary for cleaning purposes, the SLIDING BASE can be removed by removing the POSITIONING LOCK, depressing CLAMP RELEASE LEVER and sliding the BASE out of the guides. Install the BASE the reverse order.