



Instructions for Use - MPRLS-2000 Lithotomy Candy Cane Stirrups

Set Up

Slide the candy cane stirrup mounting post into an MPR-ORUC (sold separately) or an appropriately sized Clark-socket.

Use the socket, and tighten assembly at the desired height.

Test to make sure the post is properly seated and secure.

Position the patient's leg into the stirrup, ensure the strap is positioned, and secured in a manner as not to risk the patient's safety.

To make fine tuned adjustments, loosen handle on the stirrup, and move assembly to desired position. Re-tighten once desired position has been achieved.

To remove, loosen MPR-ORUC, or Clark-socket, and lift free.