



Instructions For Use MPR2718 - Knee Bolster

Product Description

The MPR Knee Bolster is a high-performance surgical positioning device designed to elevate and support the lower extremities during procedures and recovery. Made from durable, medical-grade foam and covered in premium vinyl, it features a no-zipper design that won't snag on OR table pads and allows for easier cleaning.

General Safety and Precautions

Do not use in the following situations

- Patients where leg elevation may impair circulation or pose risks
- Individuals with unstable fractures unless directed by a physician
- Situations requiring clinical judgment beyond device capabilities

Before use

- Ensure proper fit for the procedure
- Inspect for wear or damage
- Discontinue use if product integrity is compromised

Instructions For Use

Preparation

- Inspect the bolster for damage, and to verify it is clean
- Confirm correct fitment for the procedure

Positioning the Patient

- Place the ramp securely on the operating table or bed
- Gently position the patient, ensuring full support
- Align according to surgical or physician requirements

Post-Procedure

- Carefully remove patient from the bolster
- Inspect for excessive wear, damage or contamination
- Clean per facility protocol

Cleaning, Storage & Disposal

Cleaning

- Wipe with mild detergent or approved disinfectant
- Do not soak, autoclave, machine wash, or expose to high heat
- Air dry fully before reuse

Storage

- Store in a clean, dry area at room temperature
- Keep away from moisture, direct sunlight, and avoid stacking heavy items

Disposal

- Discard damaged or used components as regulated medical waste, following hospital policy and local guidelines