



MPR Instrument Makar Low-Profile Leg Holder

10228



Instructions for Use

Description

The MPR Low-Profile Leg Holder (10228) is a leg holder specially designed to enhance superior medial portal access and provide an open lateral corner to accommodate drill guides or other instruments.

It comprises a stainless steel frame with a height-adjustable top bar, and is used with a soft-cushioned foam insert (*sold separately, see ACCESSORIES*) to minimize leg rotation during procedures.

Reusable, single use, and coated foam inserts are available, in 6½" and 10" sizes. The reusable foam inserts' durable close-cell composition allows for cleaning without absorbing moisture. The single use foam inserts eliminate cross-contamination between patient and insert, and are available in a lightweight open-cell foam (blue) or a denser, firmer foam (white).

Original design by Dr. Lanny Johnson, M.D.


Indications

For use during surgical procedures to minimize leg rotation by providing quadrilateral support to the thigh.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the **CLEANING AND MAINTENANCE** section on page 3 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

Components

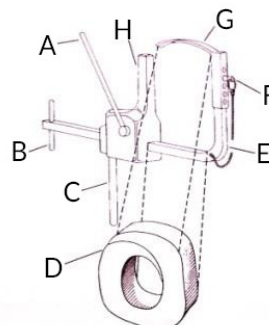

A	Cam-Action Lever
B	T-Bar
C	Post for Clark Attachment
D	Foam Insert (sold separately)
E	Medial Post
F	Height Adjustment Pin
G	Top Bar
H	Lateral Post

Accessories

REF	Description
10201	Reusable Foam (Gray)
10201C	Standard Profile 6½" Firm Reusable Spray Coated Foam Insert (Blue)
10203	XL Standard Profile 10" Reusable Foam (Gray)
10211	Standard Profile 6½" Reusable Spray Coated Foam (Blue)
10210	XL Standard Profile 10" Reusable Spray Coated Foam (Blue)
10204	Standard Profile 6½" Economy Reusable Foam Insert (White)
10205	XL Standard Profile 10" Economy Reusable Foam Insert (White)
10223B	Standard Profile 6½" Single Use Insert (Blue)
10225	XL Standard Profile 10" Single Use Insert (Blue)
10224B	Standard Profile 6½" Single Use Insert (White)
10214	XL Standard Profile 10" Single Use Insert (White)
10231	MPR Low Profile Leg Holder Replacement Bar
10234	MPR Replacement Peg and Chain
10199	T-Bar - 4" long
MPR-CS-AIO	All-In-One Clark Socket

Instructions for Use

1. With the patient on the operating room table and anesthetized, apply a tourniquet to the patient's upper thigh.
2. Ensure that the patient is situated on the table so the leg holder can be placed on the thigh 4–6 inches above the superior aspect of the patella. Place the foam insert (reusable or single use) at the same relative location.
3. Place the leg holder proximally to the foot of the table and then secure it in a Clark attachment or equivalent. Place the leg and foam insert into the leg holder framework (see Figure 1). Make certain that the indentations on either side and on the bottom of the foam insert are aligned with the framework.
4. With the thigh and foam insert properly positioned, place the top bar in position over the foam insert (see Figure 2).
5. Push the top bar down onto the medial post of the leg holder, and insert the height adjustment pin at the desired height. Keep the top bar centered over the foam insert (see Figures 3 & 4).
6. Tighten the leg holder around the foam insert and thigh by grasping the T-bar with one hand and providing a constant pulling tension, while pumping the cam-action lever up and down with your other hand. When the leg has reached a quadrilateral shape, lock in place by pushing the cam-action lever down toward the floor (see Figure 5).
7. Apply an antiseptic skin prep and drape the patient according to your facility's protocols.
8. To release the leg holder, lift up on the locked cam-action lever and push the T-bar away from your body and toward the patient.

⚠ WARNING: The leg holder has a potential tourniquet effect. Do not overtighten.

Figure 1



Figure 2

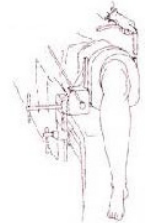


Figure 3

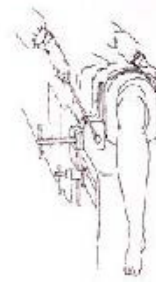


Figure 4

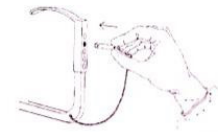


Figure 5



Cleaning & Maintenance

LEG HOLDER

The MPR Low-Profile Leg Holder framework can be draped during surgery; therefore no sterilization is required. Use the following recommendations for cleaning and decontaminating leg holders:

1. Preventing blood, saline, or other materials from drying on the framework will make cleaning easier. Do not immerse the framework for cleaning.
2. Using a soft bristle brush, manually clean the framework with an Enzol® solution prepared according to the manufacturer's recommendations. Rinse with tap water.
3. Apply a surface disinfectant to the framework with a cloth or spray. Allow to stand for the time recommended by the manufacturer.
4. Rinse the device thoroughly in lukewarm, running tap water for one minute.
5. Apply surgical lubricant or instrument milk to the leg holder framework.
6. Place the cleaned device on a sheet of absorbent matting and allow it to air dry.

FOAM INSERTS

Dispose of single-use foam inserts after use.

Wash reusable foam inserts after each use with a soapy solution and soft brush, and store where they will dry easily.

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipation profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 2 YEARS**

MPR Orthopedics Offers:

- Free trials on leg holders.
- Repair for non-working leg holders.
- Trade-ins for excess equipment.
- Reusable foam inserts (6½" and 10" sizes)
- Single use foam inserts (6½" and 10" sizes)
- Replacement top bars.
- Replacement pegs and chains
- Replacement T-bars
- Clark sockets.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative.



MPR Instrument Makar Low-Profile Leg Holder (10228)

Additional Accessories

REF	Description
MPR-ORUC	MPR Over-the-Rail Universal Clamp
MPR-CS	Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket

Also Available

REF	Description
10200MPR	MPR Instrument Makar Leg Holder
10200MPR-L	MPR Improved Original Leg Holder

A stable, time-proven orthopedic product.