



# MPR Lithotomy Candy Cane Stirrup

MPRLS-2000



## Instructions for Use



### Description

The MPR Lithotomy “Candy Cane” Stirrup (MPRLS-2000) is an device typically used in pairs to raise and hold a patient’s legs in the lithotomy position for gynecological, urological, colorectal, and other such surgeries. The assembly consists of a stainless steel cane and mounting rod, and a double-stitched cotton strap for supporting the patient’s ankle. It is designed to fit all surgical tables, and has a weight capacity of 60 pounds/27 kilograms. Its integral clamp allows for height adjustment from 23”–41”.

This device will work with a standard Clark socket or MPR Over-the-Rail Universal Clamp (*sold separately, see ACCESSORIES*).

### Indications

For holding the patient’s legs during procedures that require the lithotomy position.

### WARNING

This device can cause a number of complications if used improperly. See the INSTRUCTIONS FOR USE on page 2 for more information and preventative measures.

### Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the CLEANING AND MAINTENANCE section on page 3 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

### Components



A	Cane
B	Clamp
C	Mounting Rod
D	Ankle Support Strap


### Accessories

REF	Description
MPRLS-2020	Candy Cane Stirrups Replacement Straps (2 ct.)
MPR-ORUC	MPR Over-the-Rail Universal Clamp
MPR-CS-AIO	MPR All-in-One Clark Socket
MPR-CS	MPR Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket

### Instructions for Use

1. Attach a strap to the clip on the end of the cane.
2. Slide the candy cane stirrup mounting post into an MPR Over-the-Rail Universal Clamp (MPR-ORUC) or an appropriately sized Clark socket (*see ACCESSORIES on page 1*).
3. Secure the assembly to the table with the socket. Test to make sure the post is properly seated and secured.
4. To adjust the height of the cane, grip the pole firmly, then loosen the handle by turning it counter-clockwise. Gently raise/lower the cane to the desired height, then re-tighten the handle to secure it.
5. Position the patient's leg into the stirrup. Ensure the strap is positioned and secured in a manner so as to not risk the patient's safety.
6. Adjust position and height of stirrup as needed.
7. To remove the assembly, loosen the clamp, and lift free.

### PREVENTING COMPLICATIONS

-  If used improperly, this device can cause complications, including:
- Peroneal nerve injury
  - Compartment syndrome
  - Pressure ulcers
  - Venous stasis
  - Deep vein thrombosis (DVT)
8. In order to prevent these from occurring, make sure that enough padding is being used, and that the legs are elevated no more than is necessary.
  9. Additionally, limit the amount of time for which the patient is in the lithotomy position as much as possible.
  10. If the procedure requires the lithotomy position for an extended period of time, perform regular neuromuscular checks.

## **Cleaning & Maintenance**

### **CANE & MOUNTING ROD**

1. The cane and mounting rod can be run through a washer/decontaminator according to the manufacturer's instructions, then prepared for sterilization.
2. Alternatively, if hand washing, soak the cane and mounting rod in an enzyme solution, following the manufacturer's directions for dilution and soaking time, then rinse in warm tap water.
3. Soak or wipe the product down with a hospital approved and/or EPA approved germicide, then rinse again.
4. Wash the product with an instrument detergent up to a pH of 9.0, then rinse again.
5. Dry thoroughly.
6. Sterilize according to your facility's protocol.

### **SUPPORT STRAP**

7. Wash and sterilize according to your facility's protocol for cotton products.

## **Warranty**

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

**NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.**

**\*WARRANTY IS VALID FOR 1 YEAR**

## **MPR Orthopedics Offers:**

- Free trials on stirrup systems.
- Repairs for non-working stirrup systems.
- Replacement stirrup straps.
- Over-the-rail universal clamps.
- Clark sockets

## **For Further Information**

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative.

**A stable, time-proven orthopedic product.**