

## MPR Spring Loaded Clamp

MPR-800-0127



### Description

A clamp with a spring-loaded lever mechanism allowing it to easily be attached to and removed from a US surgical table side rail at any point. Designed to hold surgical accessories with blade-style mounting posts measuring 1" x  $\frac{3}{8}$ ".

Made in USA.

### Product Specifications

Overall Length	3" (7.62cm)
Width	2.45" (6.22cm)
Depth	1.38" (3.49cm)

### Cleaning & Sterilization

Wipe down thoroughly with a hospital- and/or EPA-approved cleaner and disinfectant prepared according to the manufacturer's instructions. If using a cleaning spray, apply the spray to a cloth and wipe instead of spraying the device directly. Avoid using phenolic or bleach-based cleaning products, or any harsh abrasives. Use another clean, dry cloth to remove the cleaning fluid, making sure that there is no pooling, then let dry.

*DO NOT* use an autoclave to sterilize this device.

### For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at [www.mprortho.com](http://www.mprortho.com).