

MPR Reusable Total Knee Pad

MPR-ECOKNEE



Description

A total knee pad made from a special biodegradable viscoelastic polyurethane foam, allowing it to be sterilized and reused. It has been rigorously tested and proven to maintain over 75% of its original integrity after 20 autoclave cycles, and breaks down up to 90% under ASTM D5511 conditions, helping hospitals and surgery centers meet sustainability goals and cut single-use pad waste by as much as 95% without any effect on patient safety.

Fits Alvarado, TK3, and all boot-style knee positioning systems, and integrates with standard instrument sets, requiring no workflow changes.

All MPR products are 100% latex-free.

Product Specifications

Length	21.75" (55.25cm)
Thickness	0.5" (1.27cm)

Cleaning & Sterilization

Between uses, remove any debris or impacted material from the pad, and sterilize in an autoclave at 135°C/275°F for 4 minutes, with a 30-minute drying period.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.