



MPR Lithotomy Candy Cane Stirrup

MPRLS-2000



Instructions for Use

Description

The MPR Lithotomy “Candy Cane” Stirrup (MPRLS-2000) is a device typically used in pairs to raise and hold a patient’s legs in the lithotomy position for gynecological, urological, colorectal, and other such surgeries. The assembly consists of a stainless steel cane and mounting rod, and a double-stitched cotton strap for supporting the patient’s ankle. It is designed to fit all surgical tables, and has a weight capacity of 60lbs (27.22kg). Its integral clamp allows for height adjustment from 23”-41” (58.42cm-104.14cm).

This device will work with all standard Clark sockets, or any of MPR Orthopedics’ universal clamps, such as the MPR Over-the-Rail Universal Clamp (*sold separately, see ACCESSORIES*).

Indications

For holding the patient’s legs during procedures that require the lithotomy position.

⚠️ WARNING ⚠️

This device can cause a number of complications if used improperly. See the INSTRUCTIONS FOR USE on page 2 for more information and preventative measures.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the CLEANING AND MAINTENANCE section on page 3 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

Components



A	Cane
B	Clamp
C	Mounting Rod
D	Ankle Support Strap

Accessories

REF	Description
MPRLS-2020	Candy Cane Stirrups Replacement Straps (2 ct.)
MPR-CS-AIO	MPR All-in-One Clark Socket
MPR-CS	MPR Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket
MPR-URC	MPR Aluminum Universal Rail Clamp
MPR-ORUC	MPR Over-the-Rail Universal Clamp


Instructions for Use

1. Attach a strap to the clip on the end of the cane.
2. Attach an appropriately sized Clark socket or other compatible clamp (see *ACCESSORIES on page 1*) to the table rail. Insert the assembly, and secure. Test for any unwanted movement.
8. Additionally, limit the amount of time for which the patient is in the lithotomy position as much as possible.
9. If the procedure requires the lithotomy position for an extended period of time, perform regular neuromuscular checks.

TO ADJUST CANE HEIGHT

3. To adjust the height of the cane, grip the pole firmly, then loosen the handle on the clamp by turning it counter-clockwise.
4. Gently raise/lower the cane to the desired height, then re-tighten the handle to secure it.
5. Position the patient's leg into the stirrup. Ensure the strap is positioned and secured in a manner so as to not risk the patient's safety. Adjust position and height of stirrup as needed.
6. To remove the assembly, loosen the socket, and lift the assembly free.

PREVENTING COMPLICATIONS

-  If used improperly, this device can cause complications, including:
- Peroneal nerve injury
 - Compartment syndrome
 - Pressure ulcers
 - Venous stasis
 - Deep vein thrombosis (DVT)
7. To prevent these from occurring, make sure that enough padding is being used, and that the legs are elevated no more than is necessary.

Cleaning & Maintenance

CANE & MOUNTING ROD

1. The cane and mounting rod can be run through a washer/decontaminator according to the manufacturer's instructions, then prepared for sterilization.
2. Alternatively, if hand washing, soak the cane and mounting rod in an enzyme solution, following the manufacturer's directions for dilution and soaking time, then rinse in warm tap water.
3. Soak or wipe the product down with a hospital approved and/or EPA approved germicide, then rinse again.
4. Wash the product with an instrument detergent up to a pH of 9.0, then rinse again.
5. Allow to dry thoroughly.
6. Sterilize according to your facility's protocol.

SUPPORT STRAP

7. Wash and sterilize according to your facility's protocol for cotton products.

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 1 YEAR**

MPR Orthopedics Offers:

- Free trials on stirrup systems.
- Repairs for non-working stirrup systems.
- Replacement stirrup straps.
- Universal clamps.
- Clark sockets.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

A stable, time-proven orthopedic product.

