



# MPR Ferkel Style Leg Holder

MPR-72201812



Instructions for Use



## Description

The MPR Ferkel Style Leg Holder (MPR-72201812) is a device typically used in conjunction with an ankle distractor and lateral post (see *ALSO AVAILABLE* on page 4) to stabilize a patient's leg during ankle arthroscopy procedures. The assembly consists of a bracket-shaped stainless steel holder plate that can be easily angled to suit the requirements of the the procedure and a 17" mounting rod for attaching to a Clark socket or equivalent (*sold separately, see ACCESSORIES*) at a wide range of possible heights.

Reusable and single-use pads for the holder plate are available (see *ACCESSORIES*). The reusable pads are spray-coated to allow for cleaning without absorbing moisture. The single use pads eliminate risk of cross-contamination between patient and pad, and are made from a lightweight open-cell foam.


## Indications

For positioning the leg during ankle arthroscopy procedures by supporting the thigh.

## Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the *CLEANING AND MAINTENANCE* section on page 2 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

## Components

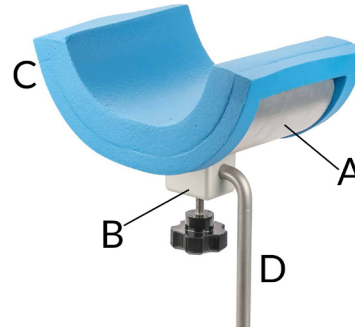

A	Holder Plate
B	Clamp
C	Foam Pad (sold separately)
D	Mounting Rod

## Accessories

REF	Description
MPR-7122	MPR Thigh Holder Reusable Pad
MPR-7123	MPR Thigh Holder Single Use Pad
MPR-7124	MPR Thigh Holder Single Use Pad (Box of 5)
MPR-CS-AIO	All-In-One Clark Socket
MPR-CS	Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket

## Instructions for Use

1. With the patient anesthetized and lying on the table in a supine position, use a lateral post to support the operative-side buttock and a non-invasive ankle distractor to distract the ankle.
2. Place a tourniquet on the patient's thigh. Inflating the tourniquet is optional unless vision is obstructed by blood during the procedure.
3. Secure the holder plate to the mounting rod, and attach a pad to the plate.
4. Use an appropriately sized Clark socket or equivalent to secure the assembly to the table and position proximal to the patient's knee.
5. Position the thigh so that the area covered by the tourniquet is resting securely on the leg holder, with the hip flexed to around 45°. Adjust position and angle of assembly and/or plate as needed.
6. Drape the patient according to your facility's protocol.

## Cleaning & Maintenance

### LEG HOLDER

The MPR Ferkel-Style Leg Holder framework can be draped during surgery; therefore no sterilization is required. Use the following recommendations for cleaning and decontaminating leg holders:

1. Preventing blood, saline, or other materials from drying on the framework will make cleaning easier. *Do not* immerse the framework for cleaning.
2. Using a soft bristle brush, manually clean the framework with an Enzol® solution prepared according to the manufacturer's recommendations. Rinse with tap water.
3. Apply a surface disinfectant to the framework with a cloth or spray. Allow to stand for the time recommended by the manufacturer.
4. Rinse the device thoroughly in lukewarm, running tap water for one minute.
5. Apply surgical lubricant or instrument milk to the framework if needed.
6. Place the cleaned device on a sheet of absorbent matting and allow it to air dry.

### PADS

Dispose of single-use pads after use.

Before cleaning spray-coated pads, inspect for any damage. If any is found, remove the pad from use.

Wipe pad down thoroughly with a neutral detergent, then disinfect with an approved germicide. Allow to dry completely. *Do not* use heat, or any harsh chemicals or abrasive materials.



## MPR Ferkel Style Leg Holder (MPR-72201812)

### Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

**NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.**

**\*WARRANTY IS VALID FOR 1 YEAR**

### MPR Orthopedics Offers:

- Free trials on leg holders.
- Repair for non-working leg holders.
- Trade-ins for excess equipment.
- Spray-coated reusable foam pads.
- Single use foam pads.
- Clark sockets.

### For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at [www.mprortho.com](http://www.mprortho.com).

---

## Also Available

REF	Description
MPR-5600	MPR Stress Post
MPR-9305	MPR Arthroscopic Stress Post
MPR RT.0010	MPR Shutt Universal Lateral Stress Post
MPR RT.0009	MPR Shutt XL Universal Lateral Stress Post
MPR RT.00010NV	MPR Shutt Universal Lateral Stress Post, w/o Velcro
MPR RT.0009NV	MPR Shutt XL Universal Lateral Stress Post, w/o Velcro
MPR014459	MPR Ankle Distractor Support Bar
MPR100213	Sterile Ankle Distractor Strap
MPR-5500	MPR Leg Holder

**A stable, time-proven orthopedic product.**