

Universal Prepper

MPR1610-008



Instructions for Use

Description

The Universal Prepper (MPR1610-008) is a device used to position and stabilize a patient's limb during prepping procedures. It is designed to work with both lower and upper extremities, is quick and easy to install and adjust, and features a U-shaped brace which can slide anywhere on the support bar and be angled to suit any and all positioning requirements. Constructed from high-grade stainless steel for durability, longevity, and easy cleaning/disinfecting.

Comes with a reusable pad, also sold separately (see ACCESSORIES).

This device will work with all standard Clark sockets, or any of MPR Orthopedics' universal clamps, such as the MPR Over-the-Rail Universal Clamp (sold separately, see ACCESSORIES).

Indications

For positioning and stabilizing the patient's limb during prepping procedures..

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the CLEANING AND MAINTENANCE section on page 2 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

Components



A	Mounting Rod/Support Bar
B	Brace
C	Clamp Handle
D	Pad

Accessories

REF	Description
MPR9510-805	MPR Reusable Universal Prepper Pad
MPR-CS-AIO	MPR All-in-One Clark Socket
MPR-CS	MPR Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket
MPR-URC	MPR Aluminum Universal Rail Clamp
MPR-ORUC	MPR Over-the-Rail Universal Clamp

Instructions for Use

1. Attach a pad to the brace of the Universal Prepper.
2. Attach an appropriately sized Clark socket or other compatible clamp (see *ACCESSORIES on page 1*) to the table rail. Insert the prepper, and secure. Test for any unwanted movement.

TO ADJUST HEIGHT

3. To adjust the height of the prepper, grip the pole firmly, then loosen the table socket.
4. Gently raise/lower the prepper to the desired height, then resecure.

TO REPOSITION BRACE

5. To adjust the position or angle of the brace, unlock the clamp by turning the handle on the bottom counter-clockwise.
6. Slide the brace along the support bar or tilt until it is in the desired position, then resecure by turning the handle clockwise.
7. Place the patient's limb into the brace. Ensure it is positioned in a manner so as to not risk the patient's safety.
8. To remove the Universal Prepper from the table after prepping is complete, loosen the socket, and lift the prepper free.

Cleaning & Maintenance

MOUNTING ROD & BRACE

1. Soak the Universal Prepper mounting rod and brace in an enzyme solution, following the manufacturer's directions for dilution and soaking time, then rinse in warm tap water.
2. Soak or wipe the product down with a hospital approved and/or EPA approved germicide, then rinse again. *Do not* use phenolic cleaners or abrasive materials.
3. Wash the product with an instrument detergent up to a pH of 9.0, then rinse again, ensuring that all cleaner is removed.
4. Allow to dry completely before storing.

PAD

Between uses, wipe pad down with a hospital- and/or EPA-approved germicide, or a **10:1** bleach solution or quaternary ammonia solution, then rinse and let dry completely. *Do not* use phenolic cleaners or abrasive materials.



Universal Prepper (MPR1610-008)

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer only.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 1 YEAR**

MPR Orthopedics Offers:

- Free trials on limb preppers.
- Repairs for defective limb preppers.
- Replacement reusable pads.
- Clark sockets.
- Universal clamps.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

Also Available

REF	Description
MPR6320-008	MPR Bilateral Leg Prepper
MPR-LP	MPR Leg Prepper
MPR-164	MPR Single Loop Leg Prepper
MPR611	MPR Picket Fence Leg Prepper
MPR611B	MPR Picket Fence Double Leg Prepper
MPR-9390	MPR Adjustable Leg Rest
MPR154	Stand-Alone Leg Prepper

A stable, time-proven orthopedic product.

