



# MPR Orange Fin Leg Holder

MPR233.061.000



## Instructions for Use



## Description

Based on a discontinued Stryker® model; the MPR Orange Fin Leg Holder (MPR233.061.000) is a device used in diagnostic and surgical arthroscopy to stabilize the leg. It is designed to accommodate a wide range of limb sizes, while allowing for easy draping of the patient and providing excellent exposure for both anterior and posterior approaches.

The device consists of an aluminum frame featuring a pair of fin-style braces on a rail/support plate, with a strap and stainless steel mounting post. Both fins are adjustable and can move along the rail with the use of a pair of locking pins. The whole frame can also move along the mounting post via the clamp on its underside.

MPR Orthopedics offers two reusable padding options for the MPR Orange Fin Leg Holder. The pads easily slip on and off, and feature a protective vinyl jacket, while the foam insert negates the need for a tourniquet. All MPR Orthopedics foam products are latex-free.

Made in USA.


## Indications

For use during diagnostic and surgical arthroscopic procedures to support the leg.

## Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the CLEANING AND MAINTENANCE section on page 3 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of the MPR Orange Fin Leg Holder before reviewing these instructions for use.

## Components

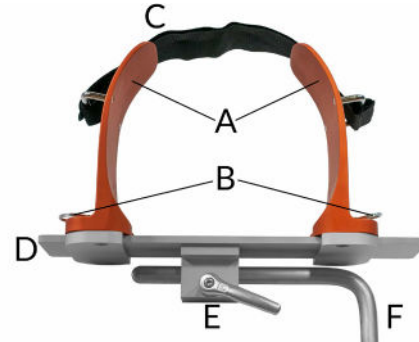

|   |               |
|---|---------------|
| A | Braces        |
| B | Locking Pins  |
| C | Strap         |
| D | Rail          |
| E | Clamp         |
| F | Mounting Post |

## Accessories

| REF         | Description                              |
|-------------|------------------------------------------|
| 233-061-100 | MPR Stryker Type Reusable Insert         |
| 233-061-200 | MPR Stryker Type Reusable Pads           |
| 233-061-300 | MPR Stryker Leg Holder Replacement Strap |
| 233-061-009 | MPR Stryker Leg Holder Pin               |
| MPR-CS      | Stainless Steel Clark Socket             |
| MPR-SRC     | MPR Over Side Rail Clark Socket          |
| MPR-CS-AIO  | All-In-One Clark Socket                  |
| MPR-ORUC    | MPR Over-the-Rail Universal Clamp        |

## Instructions for Use

### IF USING PADS

1. With the patient anesthetized and lying on the operating table in a supine position, apply a tourniquet to the patient's upper thigh if desired.
2. Attach pads to the braces, then insert the mounting rod of the leg holder assembly into the socket, and tighten the socket to secure the assembly to the table at the desired height.

### TO ADJUST FRAME

3. To adjust the position or angle of the leg holder, turn the clamp handle counter-clockwise until the frame moves on the mounting post with little resistance.
4. Move or tilt the frame to the desired position, then turn the clamp handle clockwise to secure.
5. Pull the pin of one or both braces, and open the frame enough to accept the patient's thigh. Make sure the pin is fully seated in one of the slots on the rail when released.
6. Lower the patient's leg into the frame, and move one or both braces until the pads hold the leg snugly. Tighten the strap to secure.

### IF USING INSERT

1. With the patient anesthetized and lying on the operating table in a supine position, insert the mounting rod of the leg holder assembly into the socket, and tighten the socket to secure the assembly to the table at the desired height.

### TO ADJUST FRAME

2. To adjust the position or angle of the leg holder, turn the clamp handle counter-clockwise until the frame moves on the mounting post with little resistance.
3. Move or tilt the frame to the desired position, then turn the clamp handle clockwise to secure.
4. Pull the pin of one or both braces, and open the frame enough to accept the patient's thigh. Make sure the pin is fully seated in one of the slots on the rail when released.
5. Lay the insert into the frame so that the fins hold it open and in place.
6. Lower the patient's leg into the insert, and move one or both braces until it holds the leg snugly.
7. Tuck one end of the insert into the opposite fin so that it completely encircles the leg, and tighten the strap to secure.

## Cleaning & Maintenance

### LEG HOLDER

The MPR Orange Fin Leg Holder can be draped during surgery; therefore no sterilization is required. Use the following recommendations for cleaning and decontaminating leg holders:

1. Preventing blood, saline, or other materials from drying on the device will make cleaning easier. *Do not* immerse the device for cleaning.
2. Wipe the device down thoroughly with a **10:1** bleach or quaternary ammonia solution, then rinse.
3. Apply surgical lubricant or instrument milk to the device if needed, then let dry.

### PADS

Wipe pads down after every use with a neutral detergent, then disinfect with a hospital- and/or EPA-approved germicide, and allow to dry completely before use or storage.

### FOAM INSERTS

Wipe foam inserts down after every use with a neutral detergent, then disinfect with a hospital- and/or EPA-approved germicide, and allow to dry completely before use or storage.

## Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

**NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.**

**\*WARRANTY IS VALID FOR 1 YEAR**

## MPR Orthopedics Offers:

- Free trials on leg holders.
- Repair for non-working leg holders.
- Trade-ins for excess equipment.
- Reusable pads.
- Reusable foam inserts.
- Replacement locking pins.
- Replacement straps.
- Clark sockets.
- Universal clamps.

## For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at [www.mprortho.com](http://www.mprortho.com).



## MPR Orange Fin Leg Holder (MPR233.061.000)

---

### Also Available

| REF        | Description                                 |
|------------|---------------------------------------------|
| 5371       | MPR Leg Holder                              |
| 5372       | MPR Extended XL Leg Holder                  |
| 10200MPR   | MPR Instrument Makar Leg Holder             |
| 10200MPR-L | MPR Improved Original Leg Holder            |
| 10228      | MPR Instrument Makar Low-Profile Leg Holder |
| MPR-5500   | MPR Leg Holder                              |
| MPR250     | MPR Arthroscopic Leg Holder                 |

**A stable, time-proven orthopedic product.**