



MPR Original Crank Style Leg Holder, Adjustable

6823-00A



Instructions for Use

Description

The MPR Original Crank-Style Leg Holder, Adjustable (6823-00A) is a device used in diagnostic and surgical arthroscopy to stabilize the leg. It is designed to accommodate a wide range of limb sizes, while being compact and low-profile enough to allow for easy draping of the patient and provide excellent exposure for both anterior and posterior approaches.

The device consists of an adjustable stainless steel frame, and a pair of fin-style braces, one of which can be moved laterally using a hand crank, and attaches to the table via a standard $\frac{5}{8}$ " Clark socket or equivalent.

The leg holder comes with a pair of reusable pads for the braces, with durable, fluid-impermeable coverings (6823-12-000, *pictured*). Other reusable and single-use padding options are also available (*sold separately, see ACCESSORIES*), as well as an optional under-thigh pad bracket, with corresponding reusable and single-use pads (*pictured, sold separately, see ACCESSORIES*). All MPR Orthopedics pads and inserts are latex-free.


Indications

For use during diagnostic and surgical arthroscopic procedures to support the leg.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the CLEANING AND MAINTENANCE section on page 2 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

Components

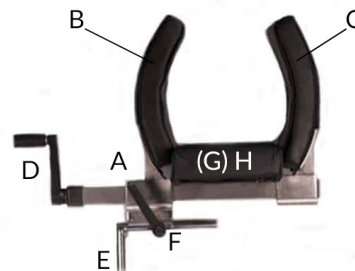

A	Frame
B	Static Brace (w/ Pad)
C	Movable Brace (w/ Pad)
D	Hand Crank
E	Mounting Post
F	Pivot/Slide
G	Under-Thigh Pad Bracket (<i>optional, see below</i>)
H	Under-Thigh Pad (<i>see below</i>)

Accessories

REF	Description
6823-10-000	MPR Original Reusable Pads
6823-10-000S	Replacement Pads for 6823 Leg Holder, Slim
6823-12-000	MPR Reusable Pads
6823-12-001	MPR Deluxe Reusable Pads
6823-11-000	MPR Reusable 3 Pc. Pad Set*
6423-14-000	MPR XL Leg Holder Pad, Single-Use
6823-15-000	MPR 3 Pc. Pad Set, Single-Use*
6823-04-000	MPR Pad Bracket
6823-13-000	MPR Reusable Under Thigh Pad
6823-06-000	MPR Reusable Under Thigh Pad w/ Velcro
6823-16-000	MPR Under Thigh Pad, Single-Use
MPR-CS	Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket
MPR-CS-AIO	All-In-One Clark Socket
MPR-ORUC	MPR Over-the-Rail Universal Clamp

*includes Under Thigh Pad

Instructions for Use

1. With the patient anesthetized and lying on the operating table in a supine position, apply a tourniquet to the patient's upper thigh if desired.
2. Slide pads onto the braces of the leg holder. If using an under-thigh pad, attach bracket and pad to the frame between the braces.
3. Attach a standard Clark socket or equivalent to the table rail so the leg holder can be positioned 4–6 inches above the superior aspect of the patella.
4. Insert the mounting rod of the leg holder into the socket, and tighten the socket to secure it to the table at the desired height.

TO ADJUST THE FRAME

5. Hold the frame firmly, then pull up on the lever on the pivot/slide. Slide and/or rotate the frame until it is at the desired position, then push the lever back down to secure.
6. Turn the crank clockwise to open the leg holder, and carefully lower the patient's leg into the braces.
7. When the patient's leg is in position, turn the crank counter-clockwise to close the holder. *Do not* over-tighten.
8. To release patient, hold the leg in place and open the holder, then gently lift leg out.

Cleaning & Maintenance

LEG HOLDER

The MPR Original Crank Style Leg Holder, Adjustable can be draped during surgery; therefore no sterilization is required. Use the following recommendations for cleaning and decontaminating leg holders:

1. Preventing blood, saline, or other materials from drying on the framework will make cleaning easier. *Do not* immerse the framework for cleaning.
2. Using a soft bristle brush, manually clean the framework with an Enzol® solution prepared according to the manufacturer's recommendations. Rinse with tap water.
3. Apply a surface disinfectant to the framework with a cloth or spray. Allow to stand for the time recommended by the manufacturer.
4. Rinse the device thoroughly in lukewarm, running tap water for one minute.
5. Apply surgical lubricant or instrument milk to the leg holder framework.
6. Place the cleaned device on a sheet of absorbent matting and allow it to air dry.

PADS

Dispose of single-use foam pads after use.

Wipe reusable pads down after every use with a neutral detergent, then disinfect with a hospital- and/or EPA-approved germicide, and allow to dry completely before use or storage.



MPR Original Crank Style Leg Holder, Adjustable (6823-00A)

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer only.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 1 YEAR**

MPR Orthopedics Offers:

- Free trials on leg holders.
- Repair for non-working leg holders.
- Trade-ins for excess equipment.
- Single use and reusable pads.
- Under-thigh pad brackets.
- Single use and reusable under-thigh pads.
- Clark sockets.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

Also Available

REF	Description
6823-00	MPR Original Crank Style Leg Holder
6823-00 XL	MPR XL Leg Holder
5371	MPR Leg Holder
5372	MPR Extended XL Leg Holder
10200MPR	MPR Instrument Makar Leg Holder
10200MPR-L	MPR Improved Original Leg Holder
10228	MPR Instrument Makar Low-Profile Leg Holder
MPR250	MPR Arthroscopic Leg Holder
MPR233.061.000	MPR Orange Fin Leg Holder

A stable, time-proven orthopedic product.

