



MPR Shoulder Suspension Kit

MPR-SS-KIT



Instructions for Use

Description

The MPR Shoulder Suspension Kit (MPR-SS-KIT) is a single-use positioning device used in conjunction with a shoulder distractor to abduct the arm during shoulder arthroscopic surgeries.

The kit is easy and quick to assemble and includes a 4" self-adhering bandage wrap, and a cloth-lined stockinette and polypropylene traction rope with S-hooks, wrapped in a sterile drape with a convenient tray. It is designed to maximize patient comfort and work with all shoulder distractors that use the S-hook design. All components are latex-free.

Made in USA. Sold in cases of 10.

Indications

For use during shoulder arthroscopic procedures to position the patient's arm.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS PRODUCT TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided sterile and is intended for single use *only*. Dispose after use.
- Prior to use, examine for possible damage and to ensure proper functioning. *Do not* use if packaging is breached or otherwise damaged prior to use.
- A surgeon *should not* attempt clinical use of this product before reviewing these instructions for use.

Components



A	Drape
B	Traction Rope w/ S-Hook
C	Tray
D	Bandage Wrap
E	Stockinette w/ S-Hook

Also Available

REF	Description
MPR-3130	MPR Premier Shoulder Holder/Distractor
MPR-3143	MPR Wishbone Shoulder Holder/Distractor System
MPR-STH	MPR Shoulder Traction Hook
MPR-STH-Z	MPR Shoulder Traction Hook, Compact
MPR0350	MPR Traction System Weight, 0.5lb
MPR0352	MPR Traction System Weight, 1.5lb
MPR0354	MPR Traction System Weight, 2.5lb
MPR0357	MPR Traction System Weight, 5lb
MPR0362	MPR Traction System Weight, 9lb
MPR0359	MPR Traction System Weight, 10lb

Instructions for Use

1. Inspect the product's packaging for signs of damage. If any are found, *do not* use.
2. Remove the contents from the packaging and unwrap, using the drape as a sterile barrier.
3. Once the patient's arm has been prepped, unroll the stockinette and slide it onto the arm up to 3"–4" (8cm–10cm) below the shoulder (*see Figure 1*).
4. Wrap the bandage tightly around the stockinette.
5. Attach the rope to the shoulder distractor, and tie the loose end to the S-hook on the stockinette. Attach a traction hook to the S-hook on the other end of the rope (*see ALSO AVAILABLE on page 1*).
6. Raise the patient's arm to an angle of 30° relative to the body (*see Figure 1*), and hold in place using traction weights (*see ALSO AVAILABLE on page 1*).

MPR Orthopedics Offers:

- Trade-ins for excess equipment.
- Shoulder distractors.
- Shoulder traction hooks.
- Traction system weights.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

Figure 1



A stable, time-proven orthopedic product.