



MPR Elbow, Hand Positioner

MPR054



Instructions for Use

Description

The MPR Elbow, Hand Positioner (MPR054) is a device used to position and stabilize a patient's arm during elbow arthroscopy procedures. The simple construction consists of a U-shaped bracket and 18.5" (46.99cm) long offset mounting post, offering a wide range of working heights, and is easy to set up and take down. Made from high-grade stainless steel for durability, longevity, and easy cleaning/disinfecting.

Comes with a reusable pad, also sold separately (see **ACCESSORIES**).

This device will work with all standard Clark sockets, or any of MPR Orthopedics' universal clamps, such as the MPR Over-the-Rail Universal Clamp (sold separately, see **ACCESSORIES**).


Indications

For positioning and stabilizing the patient's arm during elbow arthroscopy procedures.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the **CLEANING AND MAINTENANCE** section on page 2 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of this device before reviewing these instructions for use.

Components

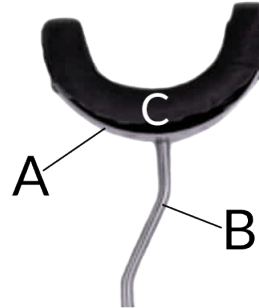

A	Brace
B	Mounting Post
C	Pad

Accessories

REF	Description
MPR054-P	MPR Reusable Replacement Pad
MPR-CS-AIO	MPR All-in-One Clark Socket
MPR-CS	MPR Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket
MPR-URC	MPR Aluminum Universal Rail Clamp
MPR-ORUC	MPR Over-the-Rail Universal Clamp

Instructions for Use

1. Attach a pad to the brace of the positioner.
2. Attach an appropriately sized Clark socket or other compatible clamp (*see ACCESSORIES on page 1*) to the table rail. Insert the positioner, and secure. Test for any unwanted movement.

TO ADJUST HEIGHT

3. To adjust the height of the positioner, grip the pole firmly, then loosen the table socket.
4. Gently raise/lower the positioner to the desired height, then resecure.
5. Place the patient's arm into the brace. Ensure it is positioned in a manner so as to not risk the patient's safety.
6. To remove the positioner from the table after the procedure is complete, loosen the socket, and lift the positioner free.

Cleaning & Maintenance POSITIONER

1. Soak the MPR Elbow, Hand Positioner in an enzyme solution, following the manufacturer's directions for dilution and soaking time, then rinse in warm tap water.
2. Soak or wipe the product down with a hospital- and/or EPA-approved germicide, then rinse again. *Do not* use phenolic cleaners or abrasive materials.
3. Wash the product with an instrument detergent up to a pH of 9.0, then rinse again, ensuring that all cleaner is removed.
4. Allow to dry completely before storing.

PAD

Between uses, wipe pad down with a hospital- and/or EPA-approved germicide, or a **10:1** bleach solution or quaternary ammonia solution, then rinse and let dry completely. *Do not* use phenolic cleaners or abrasive materials.



MPR Elbow, Hand Positioner (MPR054)

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer only.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 1 YEAR**

MPR Orthopedics Offers:

- Free trials on elbow positioners
- Repairs for defective elbow positioners.
- Replacement reusable pads.
- Clark sockets.
- Universal clamps.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

Also Available

REF	Description
MPR1610-008	Universal Prepper

A stable, time-proven orthopedic product.

