



MPR Arthroscopic Leg Holder

MPR250



Instructions for Use

Description

The MPR Arthroscopic Leg Holder (MPR250) is a device used in diagnostic and surgical arthroscopy to stabilize the leg. It is designed to accommodate legs of all sizes, while being compact and low-profile enough to allow for easy draping of the patient and provide excellent exposure for both anterior and posterior approaches.

The device requires minimal assembly and consists of a stainless steel frame with a pair of braces which can be adjusted along a support bar with 22 holes for maximal flexibility in positioning, and a Velcro strap that closes over the top.

MPR Orthopedics offers a variety of padding options, including foam and gel inserts (*sold separately, see ACCESSORIES*). The reusable foam inserts' durable close-cell composition allows for cleaning without absorbing moisture, while the gel inserts are easy to disinfect, and the single-use pads are made from lightweight, comfort-maximizing materials.


Indications

For use during diagnostic and surgical arthroscopic procedures to support the leg.

Cautions

- UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
- This product is provided non-sterile and must be properly cleaned prior to use. Refer to the **CLEANING AND MAINTENANCE** section on page 3 of this document.
- Prior to use, examine the instrument for possible damage and to ensure proper functioning.
- A surgeon *should not* attempt clinical use of the MPR Arthroscopic Leg Holder before reviewing these instructions for use.

Components

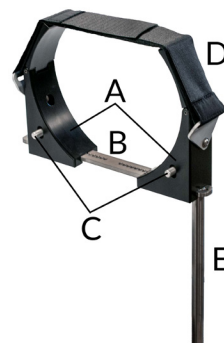

A	Braces
B	Bar/Rail
C	Locking Pins
D	Strap
E	Mounting Post

Accessories

REF	Description
MPR8002	MPR Reusable Leg Holder Insert
MPR2500	MPR Gel Reusable Leg Holder Insert
233-061-100	MPR Stryker Type Reusable Insert
3757	MPR Versatile Pad, Single Use
MPR-CS	Stainless Steel Clark Socket
MPR-SRC	MPR Over Side Rail Clark Socket
MPR-CS-AIO	All-In-One Clark Socket
MPR-ORUC	MPR Over-the-Rail Universal Clamp

Instructions for Use

1. With the patient anesthetized and lying on the operating table in a supine position, apply a tourniquet to the patient's upper thigh if desired.
2. Attach a 5/8" Clark socket or equivalent to the operating table 5–6" superior to the patient's knee.
3. Insert the mounting rod of the leg holder assembly into the socket, and tighten the socket to secure the assembly to the table at the desired height.
11. Follow your facility's protocol for draping the patient.
12. To release the patient, undo the strap, then open braces, and carefully lift leg out.

TO OPEN/ADJUST BRACES

4. Undo fastening strap.
5. Depress the locking pin on the brace which you wish to adjust and hold it down while sliding the brace along the rail.
6. Once the brace is in the desired position, release the pin. Make sure it clicks into place, then refasten the strap.
7. Wrap pad/insert around the patient's leg.
8. Open the braces wide enough to accept the leg and pad/insert, then lower the leg into the holder until it is resting against the support bar.
9. Adjust angle of leg holder as needed by loosening socket and rotating to the desired position, then retightening.
10. Once patient's leg is properly positioned, close the leg holder and fasten with the strap.

Cleaning & Maintenance

LEG HOLDER

The MPR Arthroscopic Leg Holder can be draped during surgery; therefore no sterilization is required. Use the following recommendations for cleaning and decontaminating leg holders:

1. Preventing blood, saline, or other materials from drying on the device will make cleaning easier. *Do not* immerse the device for cleaning.
2. Wipe the device down thoroughly with a **10:1** bleach or quaternary ammonia solution, then rinse with lukewarm tap water.
3. Apply surgical lubricant or instrument milk to the device if needed, then let dry.

FOAM INSERTS

Dispose of single-use foam inserts after use.

Wipe reusable foam inserts down after every use with a neutral detergent, then disinfect with a hospital- and/or EPA-approved germicide, and allow to dry completely before use or storage.

GEL INSERTS

1. Wash gel inserts with a pH-neutral detergent or enzyme solution according to your facility's protocol. Rinse any residue off thoroughly with clean tap water.
NOTE: To reduce risk of tears, avoid using abrasive materials when cleaning.
2. Store either rolled up or flat. Be mindful that the gel will match its storage temperature.

⚠ Do not sterilize gel inserts. Keep away from high temperatures.

⚠ Do not use gel inserts if punctured; dispose according to your facility's protocol.

Warranty

MPR Orthopedics products are guaranteed to be free from defects in material and workmanship for the warranty period for a particular product, *beginning from date of invoice*. Contact MPR Orthopedics for specific warranty information.

This limited warranty is restricted to repair or replacement by MPR Orthopedics, at its option, of any product found to be defective during the warranty period. Damage inflicted on a product by the user that causes it to be unsuitable for refurbishment may result in additional charges, regardless of warranty status. All warranties apply to the original buyer *only*.

In no event shall MPR Orthopedics be liable for any anticipated profits, consequential damages, or loss of time incurred by the buyer with the purchase or use of any product.

NO OTHER WARRANTY, EXPRESSED OR IMPLIED, IS GIVEN.

***WARRANTY IS VALID FOR 1 YEAR**

MPR Orthopedics Offers:

- Free trials on leg holders.
- Repair for non-working leg holders.
- Trade-ins for excess equipment.
- Single use and reusable foam inserts.
- Reusable gel inserts.
- Clark sockets.

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.

Also Available

REF	Description
10200MPR	MPR Instrument Makar Leg Holder
10200MPR-L	MPR Improved Original Leg Holder
10228	MPR Instrument Makar Low-Profile Leg Holder
MPR-5500	MPR Leg Holder
5371	MPR Leg Holder
5372	MPR Extended XL Leg Holder
MPR233.061.000	MPR Orange Fin Leg Holder

A stable, time-proven orthopedic product.