

MPR Wire Twister

MPR-JT021



Description

Also known as a Jet Wire Twister; a twister for Luque wires, used during spinal, trauma, and cardiovascular procedures including sternotomy. Its gravity-assisted design uses the weight of the patient as an anchor for the twisting action, achieved by engaging the ratchet-lock mechanism, and pulling on the knob at the end of the twister with the wire in tension. Features serrated jaws for a secure grip on the wire, and an internal spring to quickly return the instrument to its starting position.

Made from solid stainless steel for durability, longevity, and easy cleaning/sterilization.

Made in USA.

Product Specifications

Overall Length	11" (27.94cm)
Extended Length	15.25" (34.74cm)
Handle Width	4" (10.16cm)
Tip Length	1" (2.54cm)
Max. Tip Spread	0.38" (0.95cm)

For Further Information

If further information on this product is required, please contact MPR Orthopedics Customer Service at 800.524.5194 in the United States, or your authorized representative, or visit the MPR Orthopedics website at www.mprortho.com.